

HELAGO-CZ, s.r.o. Commercial Register maintained by the Regional Court in Hradec Králové Section C, File 17879 Kladská 1082 500 03 Hradec Králové 3 Company ID: 25 96 39 61, VAT: CZ 25963961

Phone: 495 220 229, 495 220 394

Fax: 495 220 154 GSM gate: 602 123 096 E-mail: info@helago-cz.cz Web: http://www.helago-cz.cz 61140 - Midline Perineal Repair Trainer Order code: **4104.61140**









Cena bez DPH

Price with VAT

1.084,16 Eur

Parameters

Obstetrics, gynecology - filter

Quantitative unit

• Detailed description

Common gynecological and obstetric actions

ks

OVERVIEW

- Anatomically accurate model for teaching perineal suturing
- Model represents a 2nd degree midline vaginal tear
- Illustration pack representing each degree of perineal trauma

REALISM

- · All essential layers of tissue can accept repeated sutures
- Soft skin with a similar drag and strength to human skin

VERSATILITY

- Can be used alongside PROMPT Flex Birthing Simulator, by placing the trainer in front, to add further realism
- Quick and easy to set up during training sessions
- Clamps are provided to secure the Midline Perineal Repair Trainer to the bench top
- Trainer is compact and easily portable

CLEANING

- Should trainer require cleaning, use a soft damp cloth on the plastic parts
- Do not get material section of this trainer wet

SAFETY

- This product is latex free
- Do not perform procedures without the trainer securely clamped to a work bench or training station
- Do not introduce liquid or gel to the trainer
- Ensure care when removing sutures using small sharp scissors or a scalpel and take care to only sever practice sutures and not integral stitching

SIMULATED PATIENT

- Perineal Repair Trainers can be used with a simulated patient
- Used in conjunction with PROMPT Flex birthing models with a simulated patient

ANATOMY

- Vaginal mucosa
- Hymenal remnants
- Posterior fourchette
- Perineal muscles (bulbocavernosus and transverse presented together)
- Perineal epidermis and dermis
- Anal sphincter and mucosa

SKILLS GAINED

- · Recognition of internal anatomy
- Tissue and needle handling
- Perform mock infiltration (administering local anaesthesia, we recommend this is done "dry", without fluid)

- Deep musculature, subcuticular and superficial suturing
- Repair of 2nd degree perineal tear
- Pill rolling technique to determine classification of injury

Do you want to print the page now? Hide some parts of page before printing



Print partially

Print the page as you see it now



Print complete page

(this box will not be printed)

Edit before printing

Hover your mouse over some part of page, a red rectangle will appear. Click it to hide the selected content of page.



Print!



Show hidden