

Plaster Model Mold (32 teeth) PRO2001- UL-MO-32

Order code: **4112.328166**



Cena bez DPH

69,00 Eur

Price with VAT

83,49 Eur

Parameters

Types of replacements

Korunky a můstky

Quantitative unit

ks

- A rubber plaster mold with the same form as the PRO2001 Series jaw model designed for making plaster models
- Most teeth models can be set in the Plaster Model Mold
- Each jaw is sold separately

How to Use:

- Lightly spray wax pattern cleaner or debubblizer to prevent air bubbles from forming
- Plant the tooth models that will be used in practice into the rubber mold
- Slowly pour the mixed plaster
- Blow air around the mold and gently remove the model
- Remove excess plaster that may have stuck to and around the tooth model