



HELAGO-CZ, s.r.o.  
Commercial Register maintained by the Regional Court in Hradec Králové  
Section C, File 17879  
Kladská 1082  
500 03 Hradec Králové 3  
Company ID: 25 96 39 61, VAT: CZ 25963961  
Phone: 495 220 229, 495 220 394  
Fax: 495 220 154  
GSM gate: 602 123 096  
E-mail: info@helago-cz.cz  
Web: http://www.helago-cz.cz

## SMI B201 - SmartMan Baby Pro

Order code: 4118.SMIB201



Information about product price on demand

Parameters

Construction

Quantitative unit

Newborn manikin

ks

### Baby CPR with Accurate Real Time Feedback

Full Body Baby Manikin (1 Month to 12 Months) with Excellent Look and Feel. Very Accurate for Even Very Low Volumes of Air. It Features SmartMan's Award Winning Real Time Coloured Bar Feedback system.

#### Accurate Feedback

Compressions - Depth, Rate, Release, Too Deep, & Hitting the Spine

Ventilations - Volume, Rate and Interval

CPR - Suitable for Two Hand Method and In Arm Method

#### Lung Sizes

Choose multiple lung capacities. Single click to set size.

## **Volume Control**

Feel how different volumes are used on different size babies. See full feedback.

## **Rate Control**

Accurate feedback for even very low volumes. See air on ventimeter as it flows into the lungs.

## **Improve Tidal Flow**

Shows in and out movement of air and proper release of the BVM.

## **Baby Features**

### **Real Time ACCURATE Feedback**

- At-a-Glance Understanding of Coloured Bars
- More Accurate than the Competition
- Covers More Parameters
- Compressions - Depth, Rate and Release
- Compressions - Too Deep and Hitting Bottom
- Ventilations - Volume and Rate of Air Flow
- Ventilations - Interval Between Ventilations
- Ventilations - Tidal Flow with Rate and Volume
- Ventilations - Pressure From Incorrect Release of BVM

### **BLS CPR Skills (AHA & ERC Guidelines 2020)**

- CPR - Supporting All Protocols: 30:2, 15:2, 3:1
- CPR - Or Set Your Own Protocol
- Ventilations - Rescue Breaths (Feedback on Volume, Rate, and Interval)
- Ventilations - Change Lung Capacity For Practice On Different Size babies
- Ventilations - Special Tutor Activity
- Ventilations - New Guidelines 2020 Activity
- Practice and Test modes

### **Quality Control Tool**

- Create User Name (password protected)
- Write Comment and Save with Results
- Save and Print Results with Names
- Print Class Lists
- Export to Excel

### **Quick & Easy To Use**

- Metronome On / Off, User Sets Rate
- Self Challenge by Turning Feedback Off
- Immediate Percentage Score
- Review Performance with Coloured Bars
- Display Chart of Results