






HELAGO-CZ, s.r.o.  
Commercial Register maintained by the Regional Court in Hradec Králové  
Section C, File 17879  
Kladská 1082  
500 03 Hradec Králové 3  
Company ID: 25 96 39 61, VAT: CZ 25963961  
Phone: 495 220 229, 495 220 394  
Fax: 495 220 154  
GSM gate: 602 123 096  
E-mail: [info@helago-cz.cz](mailto:info@helago-cz.cz)  
Web: <http://www.helago-cz.cz>

US-4 - Breast Ultrasound QA Phantom  
Order code: **4103.US4**

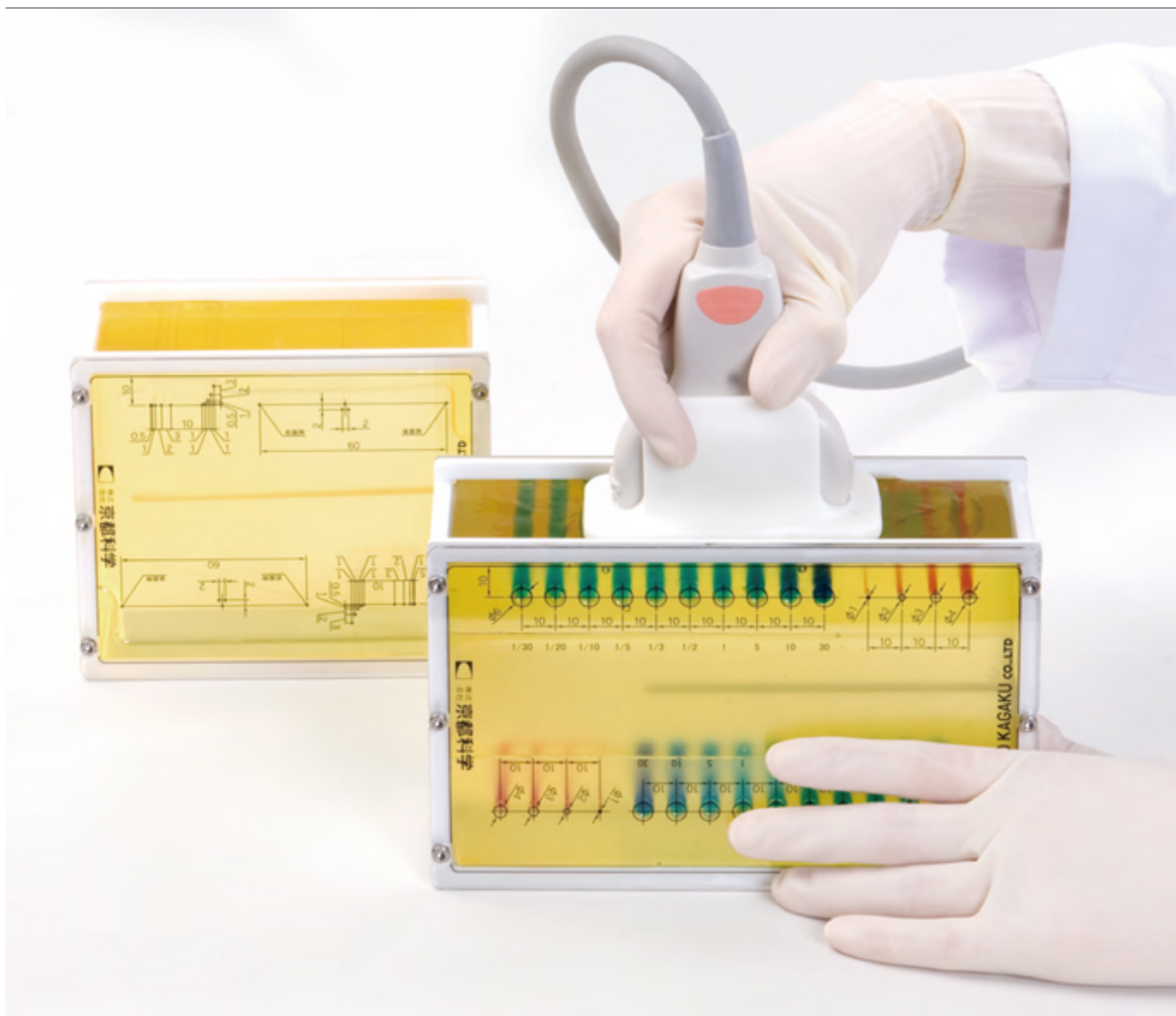

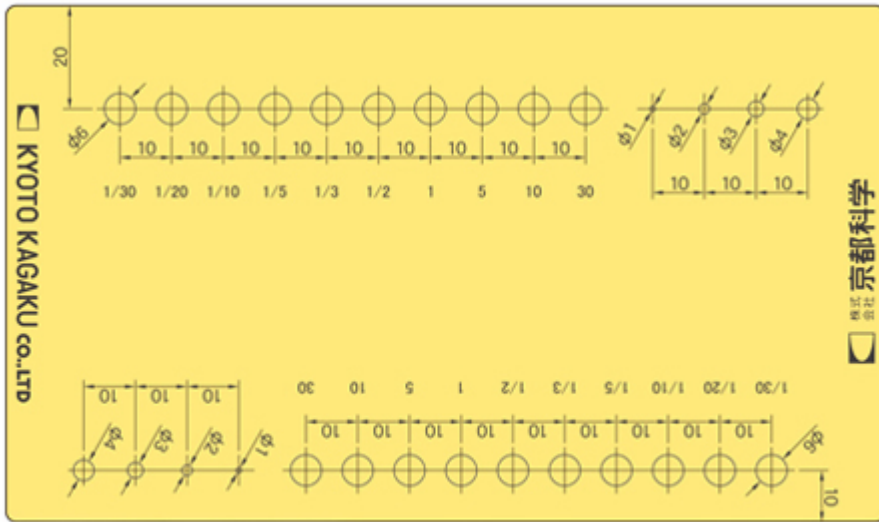

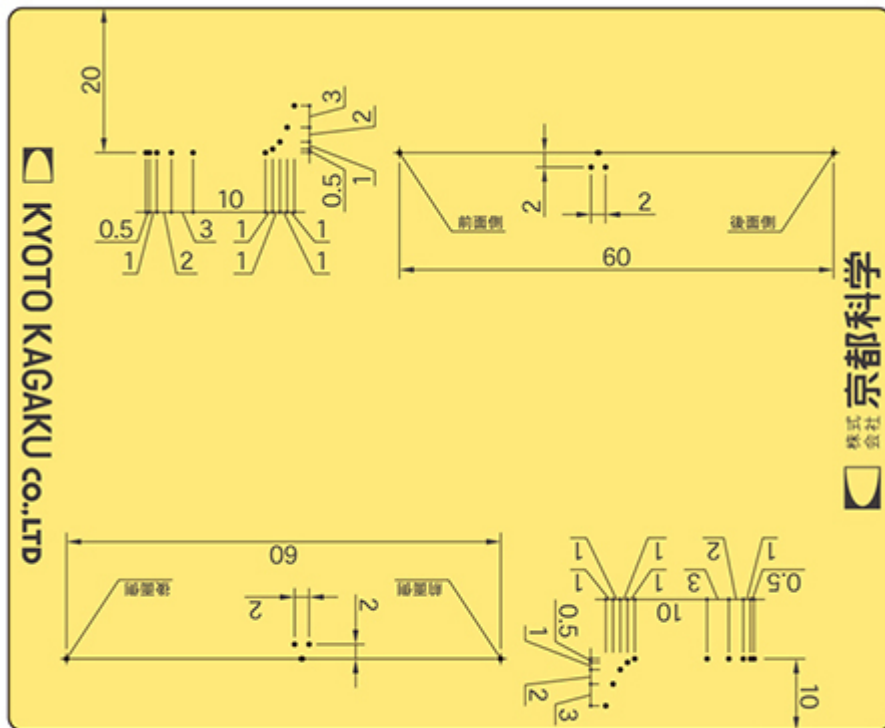

Více obrázků

Cena bez DPH

3.176,00 Eur

Price with VAT

3.842,96 Eur

Parameters

Phantoms and simulators

Ultrasound phantoms

Quantitative unit

ks

- Detailed description

Ensure highly detailed images to enable reliable breast cancer examinations.

Recommended by Japan Association of Breast and Thyroid Sonology

Ultrasound QA phantom for high precision imaging in the high frequency sonography around 10MHz required in breast examination

For monthly basic quality check of ultrasound images, as well as longer term quality assurance to maintain consistency of the performance of scanners and transducers.

## Features

- Four kinds of targets, gray scale, cyst targets, dot targets and 45 degrees line target at 2 depth, 10mm (0.4in) and 20mm (0.8in).
- Background of each phantom block is of different attenuation rate and speed of sound.
- Detailed spatial resolution as minute as 0.5mm (0.02in) can be assessed.
- Stable and durable nonaqueous phantom materials. \*Japanese patent No.3650096
- Comes with a thermometer to measure inner temperature of the phantom.

## Gray scale targets

- 10 step gray scale targets.
- Find the optimal gain to visualize all targets clearly and with even change in echogenicities.
- Then keep the gain for subsequence assessment session.

## Cyst targets

- 4 targets with different diameter from 1mm(0.04inch) to 4mm(0.16 inch).
- Visualize the four targets as clear as possible.
- Then check their roundness and if the 1mm(0.04inch) dia. target can be properly recognized.

## Dot targets

- Dot targets for horizontal and vertical resolution.
- Spacing from 0.5mm(0.02inch)to3mm(0.12inch).

## 45 degrees line target

- Influence on the image quality by the slice thickness can be assessed.
- Consists of two dot targets with 2mm(0.08inch) distance with a line target embedded horizontally with 45 degrees from the front wall to the back wall.

## Set includes

- 1 mass targets block
- 1 dot targets block
- 1 thermometer
- 1 storage case

## Size

Mass targets block phantom size:

- W18xD7.5xH11 cm
- W7.5xD3xH4.4 inch

Mass targets block phantom weight:

- 1.3kg / 2.9 lbs

Dot targets block phantom size:

- W13.5xD7.5xH11cm
- W5.4xD3xH4.4 inch

Dot targets block phantom weight:

- 1.0kg / 2.2 lbs

Do you want to print the page now?

Hide some parts of page before printing



Print partially

Print the page as you see it now



Print complete page

(this box will not be printed)

Edit before printing

Hover your mouse over some part of page, a red rectangle will appear. Click it to hide the selected content of page.



Print!



Show hidden